


CovaWound™ Hydrocolloid

Hydrocolloid Dressing

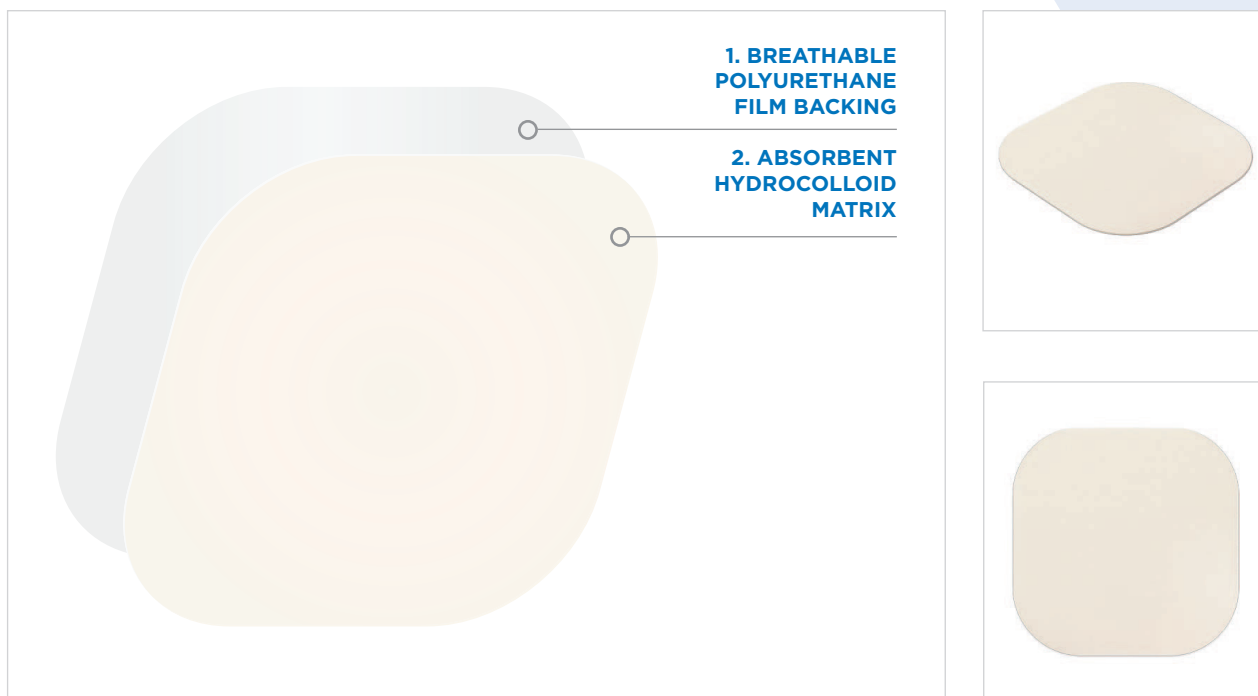

An absorbent hydrocolloid matrix which has been laminated on to a breathable polyurethane film backing.

The hydrocolloid matrix absorbs exudate to form a cohesive gel whilst the breathable polyurethane film provides a high rate of moisture vapour transmission. The combination of these features ensures that CovaWound™ Hydrocolloid provides the optimal moist wound healing environment.

A two-layer constructed dressing

1. BREATHABLE POLYURETHANE FILM BACKING

Provides a high rate of moisture vapour transmission. Acts as a barrier to external contamination.

2. ABSORBENT HYDROCOLLOID MATRIX

Absorbs exudate to form a cohesive gel that creates a moist wound environment.

Exudate level

🔴 🟡 🟢 Light

Exudate level indicates the highest level of exudate that the dressing can accommodate.

Product Codes and Sizes: Available to order through NHS Supply Chain Advanced Wound Care Framework and Alliance Healthcare

Dressing Size (cm)	Pack Size	Covalon Article Number	PIP Code	Catalogue Code
10 x 10	10	5001010	406-1024	ELM513
15 x 15	10	5001515	406-1016	ELM514
20 x 20	10	5002020	406-1008	ELM515

Indications for Use

CovaWound™ Hydrocolloid dressing is indicated for the management of lightly exuding wounds such as pressure, leg and foot ulcers, superficial partial-thickness burns, donor sites, diabetic ulcers, post-operative wounds and skin abrasions. CovaWound™ Hydrocolloid dressing may also be used on small wounds towards the end of the healing phase.

Application

- Clean the wound area with normal saline or distilled water.
- Gently dry the surrounding skin and ensure that it is free of any residue.
- Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm beyond the wound margin.
- Remove the large piece of the release liner from the dressing.
- Centre and apply the exposed part of the dressing over the wound bed.
- Avoid touching the adhesive surface to minimize risk of contamination.
- Remove the small piece of the release liner from the dressing and apply.
- Finalize application of the dressing, making sure that the edges of the dressing adhere well. This initial adhesion will improve as the dressing warms up. If necessary, use medical tape around the edges to avoid roll-up.

CovaWound™ Advanced Wound Care Range



CovaWound™
Silicone Border &
Silicone Border Lite




CovaWound™
Silicone & Silicone Lite



CovaWound™
Sacral



CovaWound™
Alginate

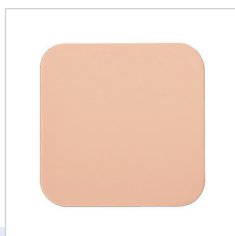

CovaWound™
Foam



CovaWound™
Adhesive Foam



CovaWound™
Hydrocolloid &
Hydrocolloid Thin/
Transparent



CovaWound™
SuperAbsorbent